RE-VO2 Video otoscope

Instruction
for use

ABOUT THIS MANUAL

READ THIS OPERATING MANUAL BEFORE ATTEMPTING TO USE THE DEVICE

This manual is valid for the device named RESONANCE RE-VO2 Video otoscope.
The edition of this manual is B/A released on 27 March 2024.

RE-VO2 is manufactured by:
M.R.S. S.r.l.
Via Cesare Battisti, 134
24025
Gazzaniga (BG)
Italy

www.resonance-audiology.com

For all inquiries, please contact us under:
sales@resonance-audiology.com for sales purposes
support@resonance-audiology.com for technical purposes and spare parts
Italy phone number: +39 035712091

IMQ Spa
Via Quintiliano, 43 20138 Milano (MI) ITALY

resonance-audiology.com

M.R.S. Srl
Via C. Battisti, 134 24025 Gazzaniga (BG) ITALY
Phone: 0039 035712091
Fax: 0039 035713966
Email: info@resonance-audiology.com

TABLE OF CONTENTS

ABOUT THIS MANUAL	2
TABLE OF CONTENTS	3
1. INTRODUCTION.....	4
1.1. THANK YOU	4
1.2. INTENDED APPLICATIONS	4
1.3. UNPACKING	4
1.4. STANDARD CONTENTS AND OPTIONAL ACCESSORIES	4
1.5. WARRANTY	4
1.6. WARNINGS.....	5
1.7. LIMITATION OF LIABILITY	5
2. IMPORTANT SAFETY INSTRUCTIONS	5
2.1. PRECAUTIONS	5
2.2. ELECTROMAGNETIC COMPATIBILITY (EMC) CONSIDERATIONS	6
3. USING THE RE-VO2.....	8
3.1. SYSTEM REQUIREMENTS:	8
3.2. SOFTWARE INSTALLATION	8
3.3. USING RE-VO2	9
3.4. SAFETY AND CLEANING	10
4. ROUTINE MAINTENANCE.....	10
5. VERIFICATION AND REPAIR OF THE INSTRUMENT	10
6. TECHNICAL SPECIFICATION	11
7. ORDERING CONSUMABLES AND ACCESSORIES	11
8. DISPOSAL INFORMATION	11

1. INTRODUCTION

1.1. THANK YOU

Thank you for purchasing a Resonance RE-VO2 Video otoscope.

This product will give many years of reliable service if treated with care.

The end user's manual will help you to get the best from the functions of your new RESONANCE RE-VO2 Video otoscope.

For supply in US only: Federal Law restricts this device to sale by or on the order of a licensed medical professional.

1.2. INTENDED APPLICATIONS

The intended application of the Resonance RE-VO2 is to perform an objective view of the outer ear before, during and after an audiological procedure.

The RE-VO2 has been designed for use by trained audiological and medical professionals in the field of audiology for examination of the outer canal and tympanic membrane. The image provided by the camera can be captured and stored as a visual record of the examination.

1.3. UNPACKING

Please check the contents of the shipping box against the contents listed below to make sure that all items ordered have been included. If anything is missing or damaged, please contact the distributor or reseller who supplied the product. (see initial page "contact details").

1.4. STANDARD CONTENTS AND OPTIONAL ACCESSORIES

STANDARD COMPONENTS		
Description	Quantity	Part number / SKU
RE-VO2 Video otoscope main unit	1	4300103520
Carrying bag	1	110000295
USB type A adapter	1	110000298
RE-VO2 Operating Manual	1	110000296BA
4mm single-use Specula	4	110000297
Alcohol pads	10	065001323

1.5. WARRANTY

All Resonance devices are guaranteed against faulty materials and manufacture. The RE-VO2 will be repaired free of charge for a period of two years from the date of installation if returned, carriage paid, to:

M.R.S. Srl
Via C. Battisti, 134
24025 Gazzaniga (BG) ITALY
Attention to: Resonance service department.

A local distributor or reseller can also be contacted.

M.R.S. will repair or replace any defective devices, fully test the system and/or components and ship the system promptly back to the owner. There is no cost for the repair or return shipping, provided the system is two years old or less and has not being misused, abused or damaged. Repair or replacement of the system as provided under this warranty is the sole and exclusive remedy of the purchaser.

M.R.S. shall not be liable for any consequential or incidental damage or for breach of any express or implied warranty. Except to the extent of applicable law, any implied warranty merchantability or fitness of this product is limited to the duration of this warranty.

M.R.S. will, at its discretion, service and repair out of warranty products at the purchaser's request, charging for parts and manpower as necessary.

1.6. WARNINGS

Throughout this manual, the following meanings of warnings and cautions apply:

WARNING

The WARNING label identifies conditions or practices that may present danger to the patient and/or user.

CAUTION

The CAUTION label identifies conditions or practices that could result in damage to the equipment.

1.7. LIMITATION OF LIABILITY

The information contained within this user's manual can be changed without prior notification.

M.R.S. accepts no liability for direct, indirect, incidental, or other damage or consequential damage arising through improper handling or through disregard of the information contained within these operating instructions.

2. IMPORTANT SAFETY INSTRUCTIONS

- The RE-VO2 is designed to be used only by professionals in the hearing healthcare industry and is intended for temporary operation.

- RE-VO2 is Class II Medical Electrical (ME) equipment that is part of ME system and provides Type B protection (Type B equipment, Type B applied parts).

- Do NOT connect items that are not specified as part of the system.
- RE-VO2 is NOT protected against water ingress.

- Power is supplied by the USB cable connected to a computer. The USB port on the computer must be able to supply at least 500mA at the standard USB voltage.

- The computer used with the RE-VO2 should conform to the requirements of IEC 60601-1 (a medical grade insulated transformer must be required).
- The RE-VO2 provides no.2 MOPP per IEC 60601-1.

- The RE-VO2 uses Type B applied parts temporarily placed on the patient ear during testing. They are nonconductive parts and can be immediately withdrawn from the patient at any time.

2.1. PRECAUTIONS

READ THIS OPERATING MANUAL BEFORE ATTEMPTING TO USE THE INSTRUMENT.

This manual is supplied as an electronic copy stored on an USB Pen drive provided with the Video otoscope.

Printed copies may also be requested at support@resonance-audiology.com

- The RE-VO2 is for indoor use only and should be used only as described in this manual.
- The RE-VO2 should be used in similar conditions to those used for audiometry and warm up in less than 5 minutes.
- The RE-VO2 must be operated on non-conductive surface only.
- Install the RE-VO2 as directed by this manual to achieve optimal use. Clean accessories per the cleaning instructions prior to use.

- The RE-VO2 is not intended to be operated in an environment with anesthetics, oxygen, or NO. It is not an AP or APG device. This ME system is NOT intended for use with flammable anesthetics.
- All components with patient contact are made of bio-compatible materials.
- The RE-VO2 does not produce any adverse physiological effects.

2.2. ELECTROMAGNETIC COMPATIBILITY (EMC) CONSIDERATIONS

The RE-VO2 Video otoscope needs special precautions regarding EMC and needs to be installed and used in accordance to the following EMC information.

List of cables and maximum length of cables, transducers and accessories:

Transducers / Accessories	Maximum length
USB cable	2 meters

- The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the RE-VO2 as replacement parts for internal components, may result in increased emissions or decreased immunity of the RE-VO2 Video otoscope.
- The RE-VO2 should not be used adjacent to or stacked with other equipment and if adjacent or stacked use is necessary, the RE-VO2 should be observed to verify normal operation in the configuration in which it will be used.
- The RE-VO2 may be interfered with by other equipment, even if that other equipment complies with CISPR emission requirements.
- The RE-VO2 does not have life supporting function.
- Portable and mobile RF communications equipment can affect the RE-VO2 Video otoscope.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the RE-VO2 Video otoscope, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Guidance and manufacturer's declaration-electromagnetic emissions		
The RE-VO2 Video otoscope is intended for use in electromagnetic environment specific below. The customer or the user of the RE-VO2 Video otoscope should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The RE-VO ² uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The RE-VO ² is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Non applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Non applicable	

Guidance and manufacturer's declaration-electromagnetic immunity			
The RE-VO2 Video otoscope is intended for use in electromagnetic environment specific below. The customer or the user of the RE-VO2 Video otoscope should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2		+/- 8 kV contact +/- 15 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4		+/- 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.

Guidance and manufacturer's declaration - electromagnetic immunity			
The Resonance RE-VO2 is intended for use in electromagnetic environment specific below. The customer or the end user of the Resonance RE-VO2 should assure that it is used in such an environment.			
Immunity test	IEC 60601 - test level	Compliance level	Electromagnetic environment - guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the Resonance RE-VO2 Video Otoscope, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:
Conducted disturbances induced by RF fields IEC 61000-4-6	3 V _{eff} 6V in ISM Bands between 0.15 and 80 MHz	3 V _{eff} 6V in ISM Bands between 0.15 and 80 MHz	
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,7 GHz 80% AM at 1 kHz	3 V/m 80 MHz to 2,7 GHz 80% AM at 1 kHz	d= 30 cm (12 inches) 80 to 800 MHz d= 30 cm (12 inches) 800 MHz to 2,7 GHz
			Were P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
Surge	+/- 1 kV for Line-to-Line +/- 2kV for line-to-ground	+/- 1 kV for Line-to-Line +/- 2kV for line-to-ground	
NOTE 1: At 80 MHz and 800 MHz, the higher frequency ranges applies.			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the RE-VO2 Video otoscope is used exceeds the applicable RF compliance level above, the RE-VO2 Video otoscope should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the RE-VO2 Video otoscope.			
^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between Portable and mobile RF communications equipment and the RE-VO2 Video otoscope			
The Resonance RE-VO2 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the end user of the Resonance RE-VO2 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the RE-VO2 Video Otoscope as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (meters)		
	150 kHz to 80 MHz d= 30 cm (12 inches)	80 MHz to 800 MHz d= 30 cm (12 inches)	800 MHz to 2,5 GHz d= 30 cm (12 inches)
0,01	0,12	0,12	0,233
0,1	0,37	0,37	0,74
1	1,17	1,17	2,33
10	3,7	3,7	7,40
100	11,7	11,7	23,3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz the separation distance for the higher frequency range applies

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

3. USING THE RE-VO2

3.1. SYSTEM REQUIREMENTS:

PC minimum requirements

- 2 GHz intel i3 processor
- 4GB Ram
- 2.5 GB available disc space
- 1024x768 resolution (1280x1024 or higher recommended)
- Hardware accelerated DirectX/Direct3D graphics card.

Supported operating system

- Windows® 10 (x64)
- Windows® 11 (x64)

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.


3.2. SOFTWARE INSTALLATION

RE-VO2 uses its own application to be installed on a PC in order to view, capture and manage images and videos. This application is called “Resoscope”.

To install Resoscope on your PC, use the installation file named “resoscope.msi” contained in the USB Pen drive supplied together with the device.

The installer file is contained inside the folder named “Resoscope”.


Execute the Resoscope.msi. Windows will warn you with the following messages.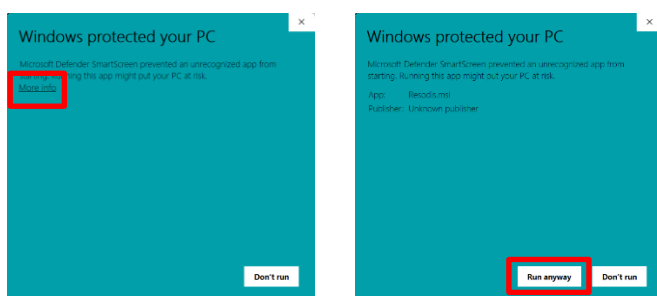

Click “More info” followed by “Run anyway”.

The Resoscope Setup Wizard will start.
Follow the instructions by pressing “Next”.

Allow Windows to install the program and at after installation press “Close” to close the Setup Wizard.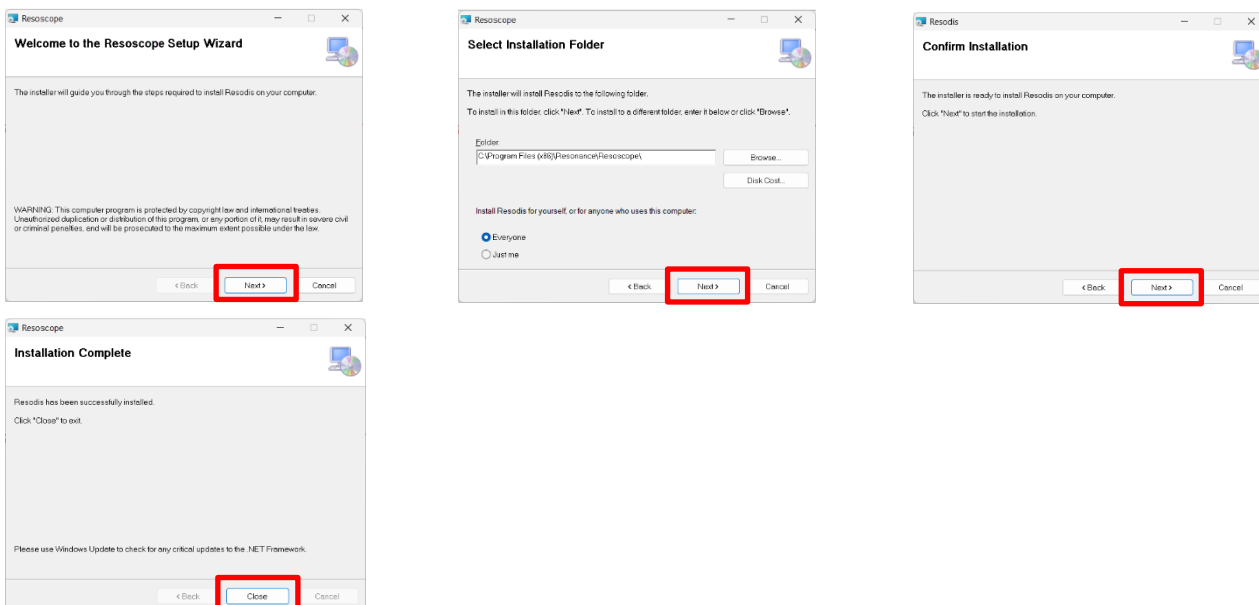


The Resoscope software is now ready to be used.

3.3. USING RE-VO2

Connect the RE-VO2 to your PC through a USB port.

After starting the Resoscope software the video otoscope will automatically be detected and switched on.

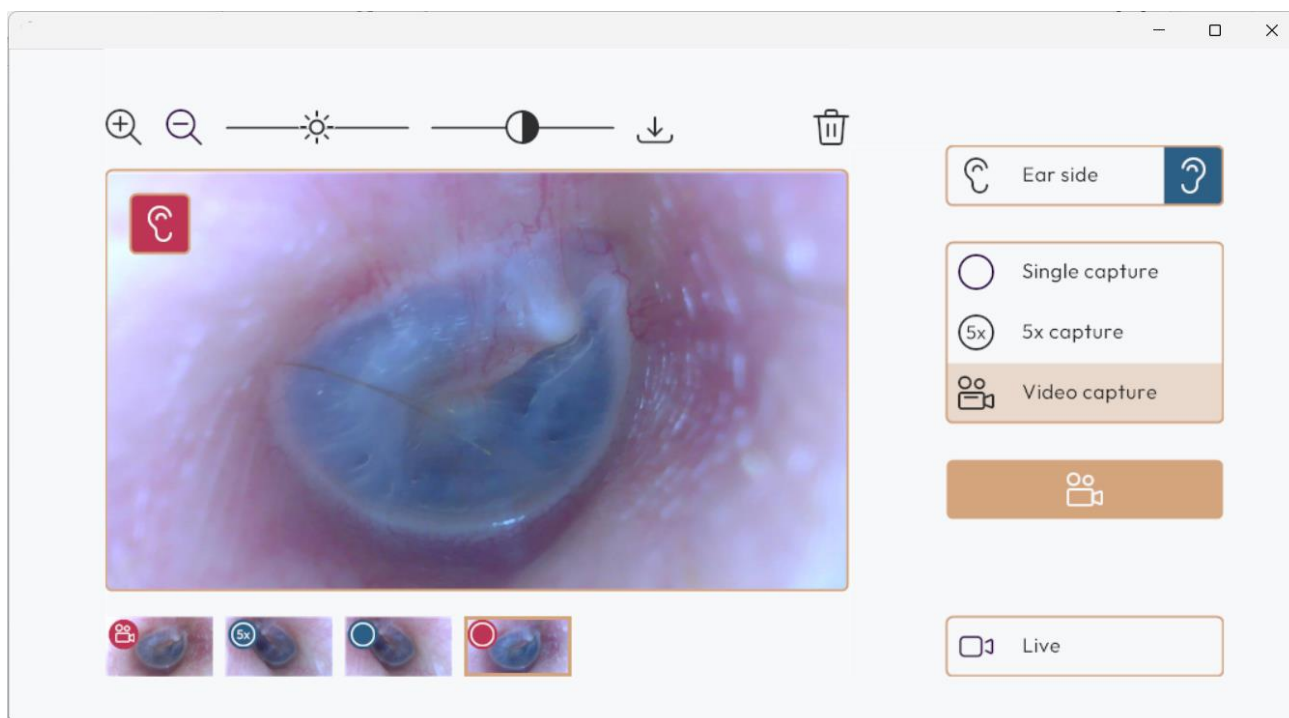Select the ear side of the ear that is inspected.

Select the preferred capturing mode: Single capture, 5x capture or Video capture.

Use the capture button on screen, the spacebar, or the button on the video otoscope to take capture an image or video.

Select one of the captures below the live image to review or edit the picture/video.

When entering edit mode, use the buttons located at the top left side of the image to perform various operations.


Use these buttons to change the zoom level, brightness and contrast level of the image.

These editing options are only available if an image and not a video has been selected, in which case only previewing, saving or deleting are available as options.

 Use the download button to save an image at a file location of your choice.
The image will be saved as shown with all the editing options active.

 Use the trash bin button to permanently delete the image/video.

To exit from editing mode and enable the capture of new images, press the button

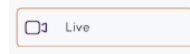Closing the Resoscope software will remove the remaining images/videos.

3.4. SAFETY AND CLEANING

The single-use speculum is always to be used with the RE-VO2.

Remove the speculum and use an alcohol pad (supplied with the unit) to wipe the video otoscope lens if the image is not clear (foggy). The speculum should always be changed between patients according to accepted medical practice.

4. ROUTINE MAINTENANCE

RE-VO2 is a precision device. Handle it carefully to ensure its continued accuracy and service.

Do NOT remove or open the camera housing, as there are not user serviceable parts.

When cleaning the instrument use a soft cloth and mild detergent to clean the RE-VO2 surfaces.

Do not allow debris or fluid to remain onto the device.

Avoid any mechanical shock to the unit.

5. VERIFICATION AND REPAIR OF THE INSTRUMENT

Please contact Resonance or the designated distributor for details of verification & services.

There are no user-serviceable parts within it.

When packing the instrument for shipping, please use the original shipping carton and packing materials or equivalent to protect against shocks the RE-VO2.

All the accessories supplied with the unit need to be returned for verification.

6. TECHNICAL SPECIFICATION

PERFORMANCE	
Image sensor:	1/9" high color sensor
Camera diameter:	4.2 mm
Pixel size:	1.4µm x 1.4 µm
Specula:	Heine 4mm
Effective picture element:	1280 (h) x 720 (v)
AGC, AES, AWB:	Automatic
Dynamic range:	68,4 dB
S/N ratio:	36,4 dB or superior
Lens construction:	3P + IR
Lens F/NO:	F4.7
Lens view angle (Field of view):	(D) 88° +/-5°
Depth of field:	10 mm – 30 mm
Light source:	High luminous white colour LED
Power input:	DC – 5V via USB type C
Output system:	Via USB type C
Power consumption:	1W
Operating temperature:	0 C° to + 40 C°
Storage temperature:	-10 C° to 70 C°
Operating relative humidity:	0% - 90% Rh
Weight/Length/Cable length:	100 g/16.8 cm/2 m
Connectivity:	To a PC via USB type C to capture, view and store images Windows® 10 and 11 compatible
Software compatibility:	Resoscope for Windows

7. ORDERING CONSUMABLES AND ACCESSORIES

To order parts, please contact Resonance (see contact details page) or the designated distributor for current prices and delivery charges.

The items available are listed below:

PART NUMBER / SKU	DESCRIPTION
C1000083	4 mm single-use Specula (box of 100 pcs.)
C1000084	Alcohol pads (box of 100 pcs.)

8. DISPOSAL INFORMATION

At the end of its service life, dispose of the RE-VO2 in keeping with all the legal requirements. The dustbin symbol indicates that waste electrical and electronic equipment may not be disposed of with normal household waste within the EU. Please use local recycling and collection points or contact the retailer where you purchased the device.

The RE-VO2 should not be discarded in the normal household waste under any circumstances. Dispose of the device via an authorized disposal center or your communal waste disposal facility. Observe the current applicable regulations. Contact your waste disposal facility in the event of any doubt. Dispose of all packaging materials via an environmentally friendly disposal facility.

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Copyright © 2024 M.R.S. Srl
All rights reserved.